Drägerwerk AG & Co. KGaA, 23542 Lübeck

**To the customers and users with
Carbon dioxide (CO₂) / oxygen (O₂)
terminal units - DIN standard**

May 2017

Important safety notice!!!

Possible gas flow on DIN CO₂ connectors in DIN O₂ terminal units

Dear Sir/Madam,

As part of our global market monitoring we have become aware of individual cases in which temporary gas flow occurred due to improper insertion of a CO₂ connector into an O₂ terminal unit by inserting with excessive force.

Due to the temporary gas flow there is the risk that O₂ instead of CO₂ is applied to a patient during a minimally invasive procedure. There were no severe patient injuries in any of the known cases.

CO₂ terminal units are installed in healthcare facilities together with O₂ terminal units in selected functional areas such as operating theatres or endoscopy. A relevant risk of confusing the terminal units only occurs where the CO₂ and O₂ Outlets are installed in direct proximity to each other.

In case of an improper insertion of a CO₂ connector into an O₂ terminal unit, the connector does not lock and is pressed out again by the gas pressure after a short period of time.

According to the applicable standard DIN 13260-2, the O₂ and CO₂ connector geometries are so similar that a CO₂ connector can also be inserted into an O₂ terminal unit. According to DIN EN ISO 9170-1, the distinctive feature of the coloured or colour-neutral version of the coupling is only the gas type symbol. According to the instructions for use, it must be ensured during application that the marking of the connector corresponds to the marking of the release sleeve of the terminal unit so that improper insertion can be avoided.

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
Postal address:
23542 Lübeck, Germany
Tel. +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
VAT no. DE135082211

Bank information:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift code: NOLADE21SPL

Company headquarters: Lübeck
Commercial register:
Municipal Court Lübeck HRB 7903 HL
General partner: Drägerwerk Verwaltungs
AG
Company headquarters: Lübeck
Commercial register:
Municipal Court Lübeck HRB 7395 HL

Chairman of the Executive Board
of
Drägerwerk AG & Co. KGaA and
Drägerwerk Verwaltungs AG:
Prof. Dr. Nikolaus Schweickart
Board of directors:
Stefan Dräger (Chairman)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske

Page 2/3

Based on the findings of the investigation carried out, Dräger has developed an improved component for the O₂ terminal unit that reliably prevents gas flow even when a current, compliant CO₂ connector is accidentally inserted using excessive insertion force. In addition, the instructions for use have been updated and now include a more detailed description of the insertion process.

During the review of the events reported in the field, we discovered that also CO₂ connectors were used that do not comply with the current standard DIN 13260-2 any more. These connectors have a smaller internal diameter of 3.5 mm which can lead to a temporary gas flow due to the tight tolerance requirements of the previous standard in connection with the plug-in coupling (even after the update). As part of an adjustment of the standard in 2004, the internal diameter of the CO₂ connector was changed to 5.9 mm.

Please check the internal diameters of all CO₂ connectors in your healthcare facility. We recommend using only CO₂ connectors that comply with the current standard DIN 13260-2.

Please contact Dräger in order to update the affected O₂ terminal units. Within the warranty period or in case of regular service by Dräger, this is done free of charge. For this purpose, please complete the attached “Customer feedback and order” form and return it to us. We will then contact you to coordinate all required activities.

Please pay attention to the following points until the upgrade is carried out:

- The gas type label of connector and terminal unit must correspond
- The coloured gas type marking (if applicable) of connector and terminal unit must correspond
- The insertion of the connector into the terminal unit must be smooth
- Pay attention to the clicking sound when the connector locks.
- Slightly pull on the connector after it has locked to check the plug connection. The connector must not loosen.
- The connector must not be pressed in when in operating position.

We regret any inconvenience caused by this and would ask you to inform all personnel in your facility accordingly. Unfortunately, this activity is unavoidable as a preventive measure to increase patient and user safety.

The competent authorities have been informed about this activity.

With many thanks in advance for your support.

Yours faithfully,

Page 3/3

Dr. V. Prapavat
Vice President Workplace Infrastructure
Product Management & Application Marketing
Drägerwerk AG & Co. KGaA

Find attached:
Customer feedback and order
Supplementary sheet for the instructions for use

cc: Chairman Medical Board and relevant Head of Departments

Supplement

Terminal units and probes DIN 13260-2

de	Ergänzung zur Gebrauchsanweisung, Seite 2	pl	Suplement do instrukcji obsługi, strona 22
en	Supplement to the instructions for use, page 4	hr	Dodatak uputama za rad, stranica 24
fr	Supplément à la notice d'utilisation, page 6	ro	Supliment la instrucțiunile de utilizare, pagina 26
es	Suplemento a las instrucciones de uso, página 8	bg	Приложение към ръководството за работа, стр. 28
it	Supplemento alle istruzioni per l'uso, pagina 10	tr	Kullanma kılavuzuna ek, sayfa 30
ptbr	Suplemento às instruções de uso, página 12		
nl	Aanvulling op de gebruiksaanwijzing, pagina 14		
lt	Naudojimo instrukcijos papildymas, 16 psl.		
lv	Papildinājums lietošanas pamācībai, 18 lappuse		
ru	Дополнение к руководству по эксплуатации, стр. 20		

WARNING

To properly use this medical device, read and comply with the instructions for use and this supplement.

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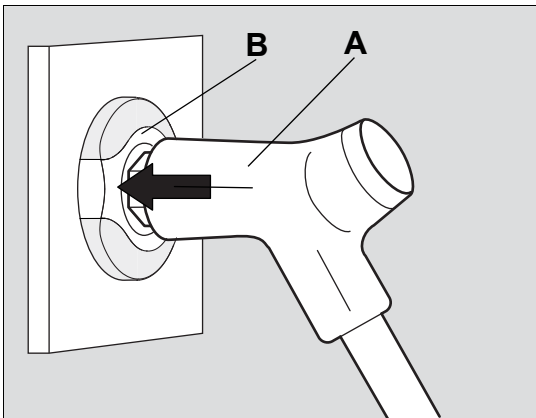
en - Supplement to the instructions for use

Keep this supplement with the instructions for use of the medical device.

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This supplement is only valid in conjunction with the instructions for use for terminal units and probes DIN 13260-2.

The supplement updates the information of the instructions for use in the following chapter:

Operation

WARNING

Risk of fire

The terminal units and probes for oxygen and nitrous oxide must not come into contact with oil, grease or flammable liquids.

WARNING

Risk of injury to the patient or user

Make sure that the gas type coding of the probe matches the gas type coding of the release bushing of the terminal unit.

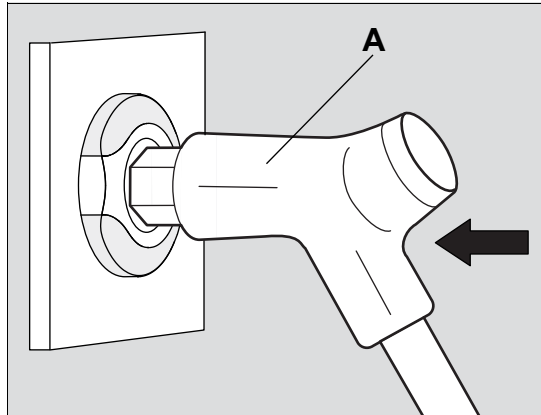
Park position

WARNING

Risk of injury to the patient or user

In the park position, check that the gas type codings correspond and check the terminal unit and probe lock.


- Press the probe (A) into the terminal unit up to the first lock position.
- Check the terminal unit and probe lock:
 - Gently pull the probe (A). The probe (A) must not come loose.
If the probe (A) can be pulled out, check the gas type coding:
 - If the gas type codings of the terminal unit and the probe (A) do not correspond, use the probe which corresponds to the terminal unit.
Then check the lock again.
 - If the gas type codings of the terminal unit and the probe (A) correspond, do not use either of them.

003


Operating position

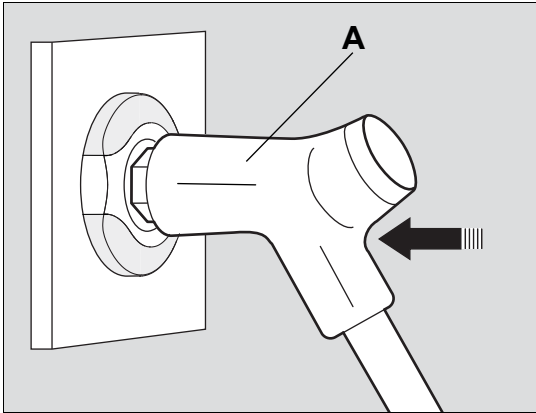- Press the probe (A) into the terminal unit up to the second lock position.


Disconnection

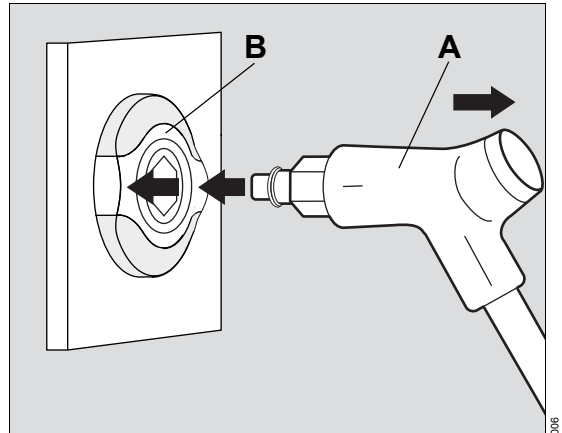- Gently press the left or right side of the release bushing (B) **and at the same time** pull the probe (A) out of the terminal unit.

Interruption of operation

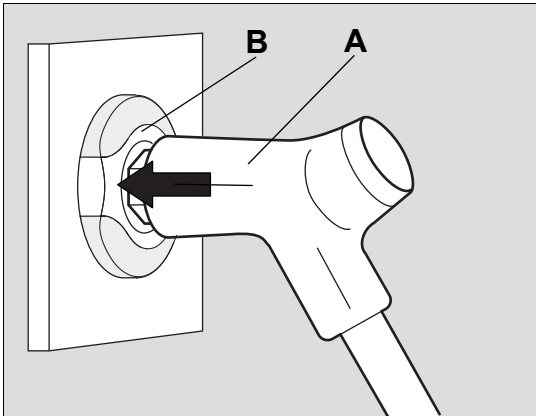- Gently press the left or right side of the release bushing (B): The probe (A) is now back in the park position (first lock position).

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© Drägerwerk AG & Co. KGaA