

PRODUCT INFORMATION

Surgical Mesh Implant TiO₂Mesh™

Dear Customer,

BioCer Entwicklungs-GmbH has been informed by a user on a potential complication using TiO₂Mesh™ surgical mesh implants. In a few exceptional cases and only under certain conditions it can happen, that after an incision parallel to the blue orientation stripes a so-called ladder is formed and a propagation of the tear without any force effect will result.

To prevent this eventuality in future, BioCer Entwicklungs-GmbH extends the instructions for use of TiO₂Mesh™ with the following warning:

“To obtain the high tear and tensile strength of TiO₂Mesh™, incisions and cavities parallel to the blue orientation stripes should generally be avoided. For indications in which an incision is required specially manufactured mesh implants are provided by the manufacturer.”

Please inform your customers actively about this amendment in the instructions for use and point out explicitly within the respective product training.

For further inquiry we remain at your disposal during our normal working hours at

customerservice@biocer-gmbh.de.

Regards

Dr. Markus Heinlein
Managing Partner