

FIELD SAFETY NOTICE

FRED easy

Your FRED easy is equipped with an internal battery, independent of the main battery (blue), which must be replaced every six years.


This is described in the user guide, in the section "Maintenance".

**The replacement of this battery every 6 years is compulsory,
the proper function of your device depends of it.**

If this instruction is not observed, the device could become unable to start.


This requires the intervention of qualified personnel, the disassembly is required.

If your FRED easy is less than 6 years old, please forecast this maintenance accordingly.

If your FRED easy is over 6 years old, and if the internal battery has not been replaced, please contact your local SCHILLER distributor, or your service provider, if you have a maintenance contract. Waiting this overdue maintenance, you can leave your defibrillator in service, controlling availability, weekly, by turning on with knob 

Begins the serial number of your FRED easy with 58xxxxxx (8 digits) or is your Serial number 058xx7xxxxxx, 058xx8xxxxxx, 058xx9xxxxxx (12 digits):
the maintenance interval has been reached and there is an urgent need for maintenance.

Wissembourg, december 22, 2015

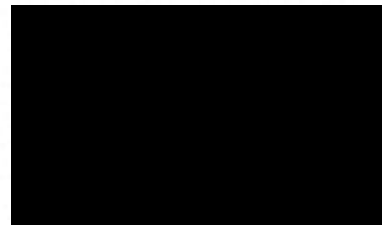

Alain Weissinger
Quality Manager



THE ART OF DIAGNOSTICS