

First Crush Pill Crusher



Do not leave to charge for more than 6 hours

If your unit needs to be recharged, the red light will blink

The unit is fully charged when the red light turns off

Step-by-Step Instructions



1 Press **ON** button to turn on unit (**GREEN** light should be on)



2 Place pills into bottom cup and put top cup on top (max. **10** pills)



3 Push **"OPEN"** button and lift lid



4 Place cups into cup holder and firmly close lid until it latches



5 Push **"EXTRA GRIND"** button



6 Remove both cups and tap **TOP** cup to remove residue



Break harder/bigger pills into half before crushing


Determine if powder is fine enough, otherwise can press "EXTRA GRIND" again

How to Identify Top and Bottom Cups

A TOP CUP AND A BOTTOM CUP ARE REQUIRED FOR EVERY CRUSH



Top cup
(ridges on
top of rim)

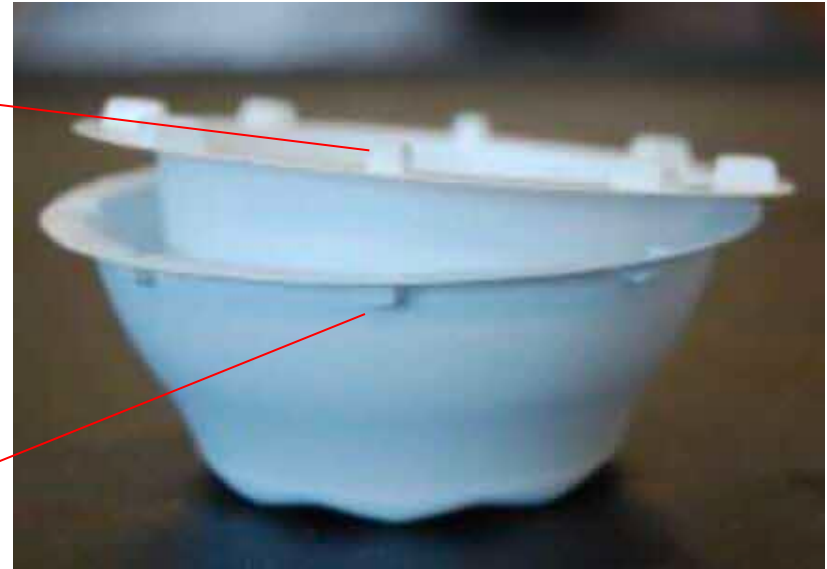

Bottom cup
(ridges on
bottom of
rim)

Complete RX cups dimensions:

Width: 6.35cm

Height: 2.16cm

Capacity: 41.25ml

Maintenance of Pill Crusher



Do not remove label.

Do not force the lid shut, check placement of disposable cups.

Remove the medication cup holder for cleaning only. **DO NOT DISPOSE.**

Hold machine upside down with lid open, tap bottom of machine to remove excess residue.

Clean daily with 75% alcohol and **WIPE DRY WITH PAPER TOWEL TO PREVENT DISCOLORING.**